


TRANSGLIDE "PULL SHEET" Pull sheet for patient transport

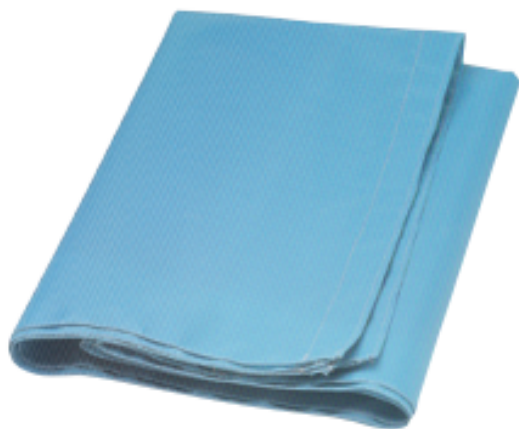

Product description:

How to use ?

- 1) Turn the patient to the side using the «pull sheet», push the Transglide under the sheet and return the patient back on to the Transglide. Hold the «pulling sheet» with both hands and keep the back straight and upright.
- 2) Using the sheet, slide the patient over the Transglide, maintaining an even motion and keeping the back straight. A second person will assist by pushing gently.
- 3) Once the patient has been placed in the correct position on the bed / couch / X ray table etc, turn him or her to the side slightly. The second assistant pulls the Transglide from the bottom end, easing it slightly, thus removing it from under the patient.

Highlights :

- Easy to use by one or two care workers
- Easy to clean and acid resistant
- X-ray translucent
- Extremely good value for money
- Size : 150 x 150 cm / 60 x 60 inch

Reference: -