

NUCLEAN - SURFACE DISINFECTION WIPES
Pre-impregnated wipes for surface disinfection in medical environments



Product description:

NUCLEAN is a **surface disinfection solution** designed for medical, clinical and hospital environments. **NUCLEAN disinfectant wipes** are suitable for cleaning and disinfecting **non-immersible medical device surfaces**, without the need for rinsing (except food-contact surfaces).

Ready to use, **NUCLEAN** wipes combine **cleaning and disinfecting action** with fast drying and no residue. Their formulation is **hypoallergenic** and designed for regular use in healthcare settings.

Main features

- Ready-to-use pre-impregnated wipes
- Suitable for non-immersible medical device surfaces
- No rinsing required (except food-contact surfaces)
- Fast drying, streak-free
- Hypoallergenic formulation
- Complete disinfectant activity within 5 minutes

Packaging

- Dispenser box of **120 impregnated wipes**
- Wipe dimensions: **190 x 130 mm**

Microbiological activity - NUCLEAN wipes

Bacteria

- EN 13727: active in 1 minute
- EN 1276: active in 5 minutes
- EN 13697: active in 5 minutes
- EN 14561: active in 5 minutes

- EN 16615: active in 5 minutes

Yeasts / Moulds

- EN 13624: active in 1 minute
- EN 1650: active in 5 minutes
- EN 13697: active in 5 minutes
- EN 14562: active in 5 minutes
- EN 16615: active in 5 minutes

Viruses

- EN 14476: active within 30 seconds to 5 minutes depending on strains

Spores

- EN 13704: *Clostridium difficile* - active in 15 minutes

Directions for use - NUCLEAN wipes

- 1 Remove one wipe from the dispenser.
- 2 Thoroughly clean the surface.
- 3 Allow to act for **5 minutes** (15 minutes for *Clostridium difficile*).
- 4 Do not rinse (except food-contact surfaces).
- 5 Use one wipe per device.
- 6 Close the dispenser after use.

Reference: AL4004