

DR. ARABIN HYBRID SOFT PESSARY – UTERINE PROLAPSE STAGE II-III, SENSITIVE MUCOSA
Soft silicone hybrid pessary for moderate uterine prolapse and sensitive vaginal mucosa



Product description:

The **Dr. Arabin Hybrid Soft pessary** is designed for the management of **grade II-III uterine prolapse** and certain forms of **incontinence**, particularly in patients with a **sensitive vaginal mucosa**. The transparent *Soft* version is indicated for normal physical activity, after surgery or childbirth, and for less pronounced prolapse.

Hybrid pessaries represent a specific development aimed at improving patient tolerance and autonomy. Their **egg-shaped contour with concave suction areas** provides better retention than conventional pessaries without widening the vagina. The device combines two mechanical effects: anterior support on the symphysis and lateral stabilization through recesses integrated into the wall.

The design allows **central drainage of discharge or blood** and simultaneous use of tampons. This pessary was developed for early pelvic floor disorders related to levator muscle lesions associated with an enlarged vaginal hiatus. Tensile force measurements showed a holding capacity approximately **five times higher** than that of conventional models.

The Hybrid Soft pessary is intended to **support pelvic organs** and relieve pressure and incontinence symptoms, including during pregnancy and the postpartum period, or when other pessaries are poorly tolerated or insufficiently retained.

The maximum uninterrupted wearing time is **30 days (Class IIa)**. However, it is recommended to remove the pessary every evening and reinsert it in the morning. The patient should be informed by a healthcare professional about use, changing and any concomitant therapy. The device is intended for **single-patient use** only. Before each insertion, the surface should be inspected; if cracks or discoloration are observed, the pessary must be replaced.

The pessary should be cleaned under running water, **without the use of disinfectant**. To ensure hygiene during self-management, a storage box may be recommended.

Available sizes

Longitudinal diameters: 55 mm / 60 mm / 65 mm / 70 mm / 75 mm / 80 mm

Pessary size: Size 2 - Ø55 mm, Size 3 - Ø60 mm, Size 4 - Ø65 mm, Size 5 - Ø70 mm, Size 6 - Ø75 mm, Size 7 - Ø80 mm

Reference: -