

## ANIOS'CLEAN EXCEL D - CLEANING AND PRE-DISINFECTANT SOLUTION

### Concentrated cleaning and pre-disinfection solution for medical instruments



#### Product description:

**ANIOS'CLEAN EXCEL D** is a concentrated solution designed for the **cleaning and pre-disinfection of medical instruments**. Its non-corrosive formulation is compatible with a wide range of materials and ensures effective treatment prior to sterilisation.

This pre-disinfectant cleaner is recommended for **surgical and medical instruments, thermosensitive devices and endoscopy equipment**. It is used by immersion, at the required dilution, as part of professional reprocessing protocols.

ANIOS'CLEAN EXCEL D offers **complete microbiological activity**, including bactericidal, yeasticidal, tuberculocidal and virucidal action. Concentration and contact time must be adapted according to the desired level of efficacy.

- **Concentrated solution** for professional use
- **Cleaning and pre-disinfection** before sterilisation
- **Compatible** with many materials
- **Immersion use**
- **Appearance:** clear blue solution
- **Available formats:** 1 L or 5 L container

#### Directions for use:

Use diluted, with a minimum contact time of **5 minutes**. Fully immerse the medical device and cover the container during contact time. Brush if visible soil is present. Rinse thoroughly with tap water after treatment. Replace the solution at least once daily or whenever visibly soiled.

Packaging: 1 L container, 2 × 5 L containers with pump

Reference: -