

ADULT POSITIONING PADS - CHEST ROLLS Pads in cross-linked viscoelastic gel



Product description:

It can be used as an abdomen support during all types of surgery.

Materials

It is made of an amorphous polyurethane gel with a density of 0.99 gr/cm³.

Maintenance and Cleaning

- The gel positioners can be easily cleaned with any cleaning agent and / or disinfectant for the operating room.
- Avoid immersing the gel positioners in the detergent and / or disinfectant for a long time.

Storage and Disposal

- Keep the positioners, if possible, on a level surface.
- Avoid placing heavy or sharp objects on the positioners.
- Keep the gel temperature between -18 and 50 degrees.
- Keep away from sources of light, moisture and heat.
- Dispose of the device in compliance with current regulations on environmental protection and separate waste collection. Package 1 Piece.

Package

1 piece

Size of gel positioning wedges 1: Choose an option, 30 x 15 x 10 cm, 40 x 15 x 10 cm, 50 x 15 x 10 cm, 30 x 10 x 8,5 cm, 40 x 10 x 8,5 cm, 50 x 10 x 8,5 cm, 50 x 18 x 15 cm



MEDICAL EQUIPMENT, ACCESSORIES AND CONSUMABLES FOR MEDICAL IMAGING.

Reference: CDP0005