

ECHO CLEAN - DISINFECTION WIPES LEVEL 2
Pre-impregnated wipes for intermediate-level disinfection of ultrasound probes



Product description:

ECHO CLEAN wipes are designed for the **cleaning** and **intermediate-level disinfection** of **ultrasound probes**. They are suitable for use on **endocavitary probes**, enabling intermediate-level disinfection in accordance with SF2H protocols, using one wipe per step (cleaning and disinfection).

ECHO CLEAN is tested and approved by ultrasound system manufacturers. Its **non-corrosive** formula is suitable for use on sensitive medical equipment regularly exposed to disinfectants.

- **Total activity within 5 minutes**
- Formula specifically developed for ultrasound probes
- Aldehyde-free and dye-free
- Fast drying
- No residue
- Ready to use
- **Dispenser box of 120 wipes, 190 × 130 mm**

Manufacturer certifications

ECHO CLEAN is certified for ultrasound probe disinfection by the following manufacturers:

- GE
- Fujifilm Sonosite
- Siemens Healthineers

Instructions for use

- 1 Remove one wipe from the container.
- 2 Thoroughly wipe the surface.
- 3 Allow the product to act for **5 minutes (15 minutes for *C. difficile*)**. Do not rinse (except for food-contact surfaces).

- 4 Use a new wipe for each device.
- 5 Close the lid carefully after use.

Disinfection level: Level 2 (high)

Reference: -