

ECHO CLEAN - DISINFECTANT SPRAY LEVEL 2
Spray for intermediate-level disinfection of ultrasound probes



Product description:

ECHO CLEAN spray is a ready-to-use solution designed for the **cleaning** and **intermediate-level disinfection** of **ultrasound probes**. It is suitable for use on endocavitary probes, in accordance with **SF2H protocols**, using a clean non-woven wipe for each step.

Tested and approved by ultrasound system manufacturers, ECHO CLEAN features a **non-corrosive formula** suitable for sensitive medical equipment regularly exposed to disinfectants.

- **Total activity within 5 minutes**
- Formula specifically developed for ultrasound probes
- Aldehyde-free and dye-free
- Fast drying
- No residue
- Ready to use
- **1-liter spray bottle**

Manufacturer certifications

ECHO CLEAN is certified for ultrasound probe disinfection by the following manufacturers:

- GE
- Fujifilm Sonosite
- Siemens Healthineers

Directions for use

- 1Pre-impregnate a clean non-woven wipe with the solution.
- 2Thoroughly wipe the probe surface.
- 3Allow the product to act for **5 minutes (15 minutes for *Clostridium difficile*)**.
- 4Do not rinse (except for food-contact surfaces).

5Use a new wipe for each device.

Disinfection level: Level 2 (high)

Reference: -