

SEPTISPRAY 2 - LOW-LEVEL SURFACE DISINFECTION SPRAY
Alcohol-free solution for low-level disinfection of medical devices



Product description:

SEPTISPRAY 2 is an **alcohol-free detergent disinfectant solution** intended for the **low-level cleaning and disinfection** of reusable medical device surfaces, both invasive and non-invasive.

Designed to preserve sensitive materials, SEPTISPRAY 2 is suitable for **wipe-based disinfection protocols** in medical practices, imaging departments and clinical environments. Its microbiological activity complies with applicable standards while ensuring broad material compatibility.

Main features

- Alcohol-free detergent disinfectant
- Low-level disinfection by wiping
- Suitable for reusable invasive and non-invasive medical devices (Class IIa)
- Wide material compatibility thanks to low surface tension
- Colourless liquid, fragrance-free
- pH 6.5–7.5
- Ready to use

Microbiological properties (dirty conditions, 20 °C)

Bacteria

- EN 1276 / EN 13727 / EN 16615: effective in **2 minutes**

Yeasts

- EN 1650 / EN 13624 / EN 16615 on *Candida albicans* and *Candida auris*: effective in **2 minutes**

Moulds

- EN 13624 / EN 17387 on *Aspergillus fumigatus*: effective in **30 minutes**

Mycobacteria

- EN 14348 on *Mycobacterium terrae*: effective in **30 minutes**

Viruses

- Enveloped viruses EN 14476 / EN 16777: effective in **2 minutes**
- Norovirus and Rotavirus EN 14476 / EN 16777: effective in **15 minutes**
- Adenovirus type 5 EN 14776: effective in **45 minutes**

Ultrasound probe compatibility

The use of **SEPTISPRAY 2 on ultrasound probes** must comply with the **manufacturer's recommendations** and the internal protocols of the healthcare facility.

Directions for use

- 1Spray the product onto the surface or onto a clean non-woven wipe.
- 2Clean the surface thoroughly.
- 3Allow the product to act according to the required contact time.
- 4Do not rinse unless specifically required.
- 5Use a new wipe for each device.

Regulatory information

- Medical device **Class IIa** (EU Regulation 2017/745)
- Professional use only
- Non-sterile product
- Latex-free, phthalate-free, no animal or biological ingredients
- Storage conditions: +5 °C to +25 °C, protected from frost and heat
- Shelf life: **23 months** from date of manufacture

Packaging

- 750 ml spray bottle
- Reference: **SP2-750**

Disinfection level: Level 1 (low)

Reference: SP2-750