

## OVAL PATIENT POSITIONING PAD Gel Positioner for Operating Table.



### Product description:

This extremely versatile positioners are good for any procedure. Beneficial for tracheotomy, thyroidectomy, tonsil and adenoid cases, cataract or other ophthalmic procedures.

#### Materials

It is made of an amorphous polyurethane gel with a density of 0.99 gr/cm<sup>3</sup>.

#### Maintenance and Cleaning

- The gel positioners can be easily cleaned with any cleaning agent and / or disinfectant for the operating room.
- Avoid immersing the gel positioners in the detergent and / or disinfectant for a long time.

#### Storage and Disposal

- Keep the positioners, if possible, on a level surface.
- Avoid placing heavy or sharp objects on the positioners.
- Keep the gel temperature between 50 ° and -18 °.
- Keep away from sources of light, moisture and heat.
- Dispose of the device in compliance with current regulations on environmental protection and separate waste collection.

#### Package

1 Piece.

Size of gel positioning wedges 1: Choose an option, 16 x 8 x 4.5 cm, 33 x 7 x 5 cm, 33 x 12 x 6 cm, 36 x 12.5 x 7 cm, 25 x 12.5 x 5 cm, 38 x 12.5 x 6.3 cm, 50 x 12.5 x 7 cm

Reference: -